


# Patient Test Instructions

Please read these instructions before you start your hearing test.

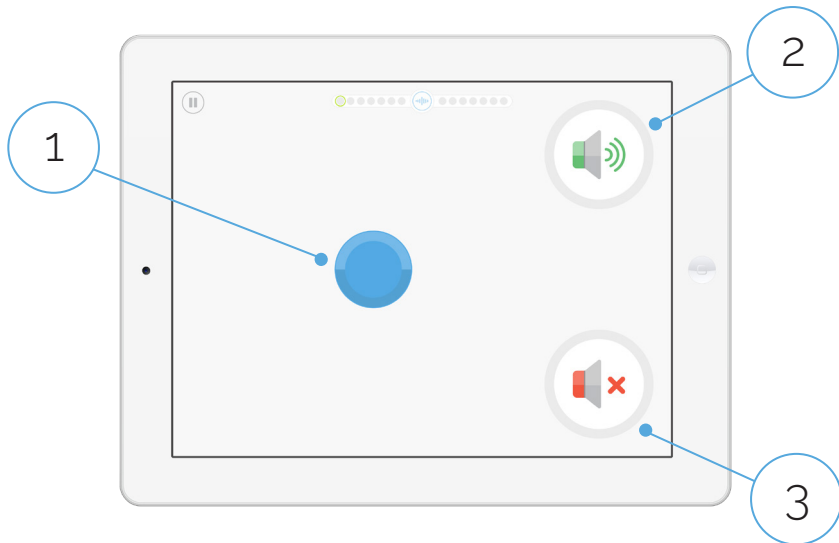

1. Press and hold the blue disc to present a sound
2. If you **hear a sound** (even a very quiet one), tap or drag the blue disc to the green “sound speaker” icon
3. If you **don’t hear any sound at all**, tap or drag the blue disc to the red “no sound speaker” icon
4. Continue pressing and sorting the blue disc until the test is complete

## Special Notes

- More than half the time you will not hear a sound when you press the blue disc. This is normal, and does not mean you’re doing the test wrong or have hearing loss
- You may or may not hear a sound. You can press the blue disc as many times as needed if you’re unsure if you heard a sound